


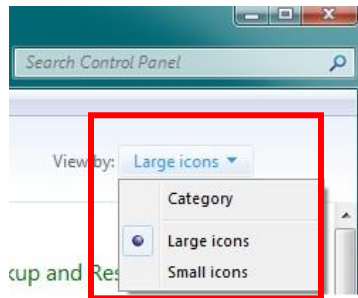
Configuring Windows 7 for Sierra Wave

This document will describe how to setup Windows 7 for optimal performance of the Sierra Wave program, please configure Window 7 as outlined in the following instructions.

Cadwell recommends Windows 7 Professional or Windows 7 Ultimate.

1. Turn off the UAC (User Account Control) before installing Sierra Wave:

- Click the **Start Orb** .
- Click on the **Control Panel**.
- Change the “**View by:**” to “**Large Icons**” or “**Small Icons**”. *Do not use “Category View”.*



- Then click on **User Accounts**.
- In that dialogue box click on “**Change User Account Control Settings**”. Answer “**Yes**” if a prompt is displayed.


Make changes to your user account


[Change your password](#)

[Remove your password](#)

[Change your picture](#)


 [Change your account name](#)

 [Change your account type](#)

 [Manage another account](#)


 [Change User Account Control settings](#)


- Move the slider to the “Never Notify” position. Click **OK**.

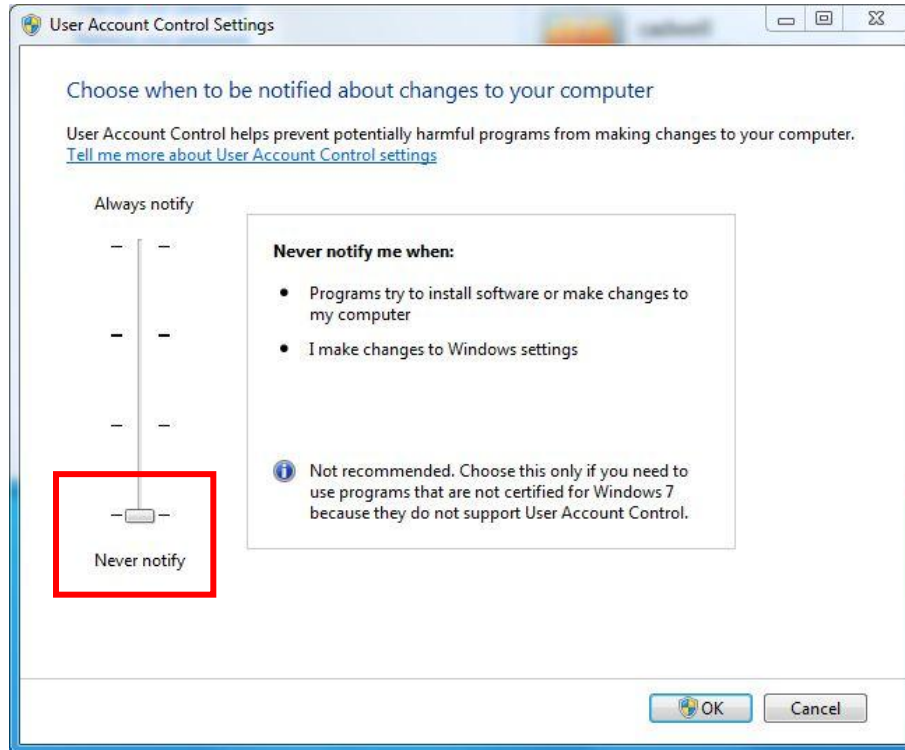


- **Restart** the computer.

2. Configure the Network Card:


- Click the **Start Orb** .
- Click on the **Control Panel**.
- Change the “**View by:**” to “**Large Icons**” or “**Small Icons**”. *Do not use “Category View”.*
- Click on “**Network and Sharing Center**”.

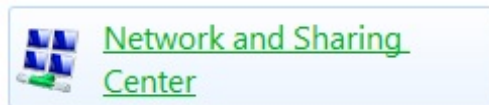


- Then click on “**Change Adapter Settings**” on the left hand task pane. This will show all the network connections available on the computer.

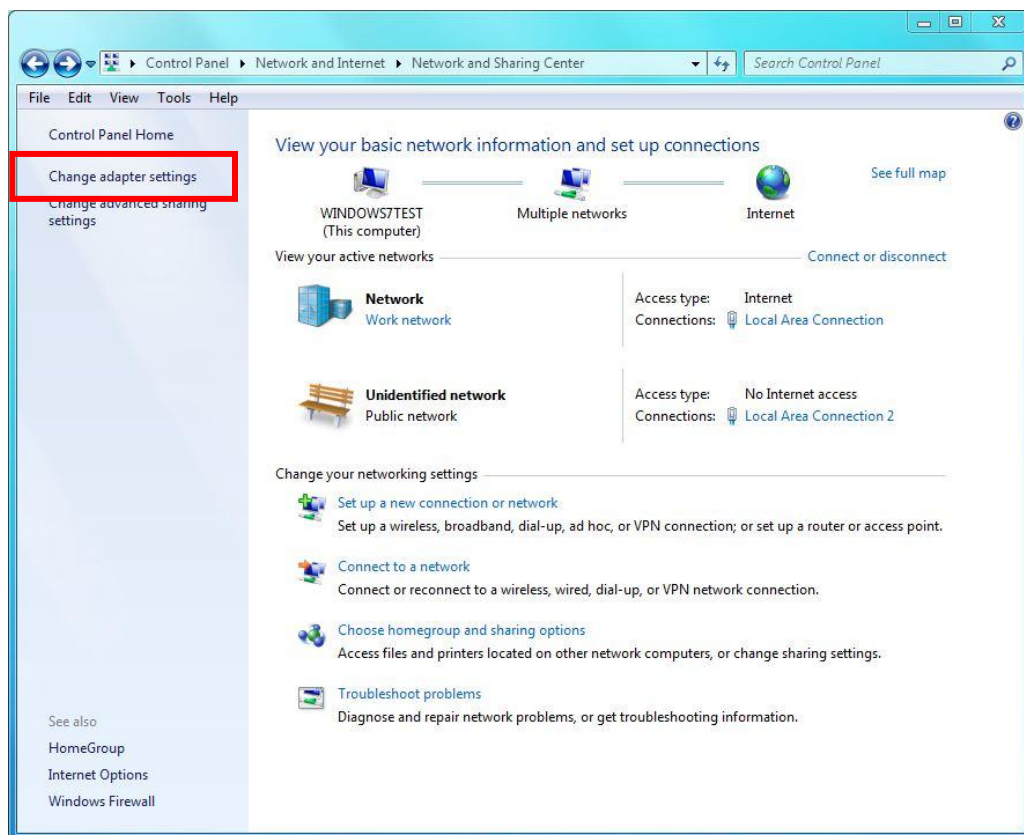


- **Right-click** on the LAN or High-Speed Internet connection that will be used to communicate to the base unit and select **“Properties.”**

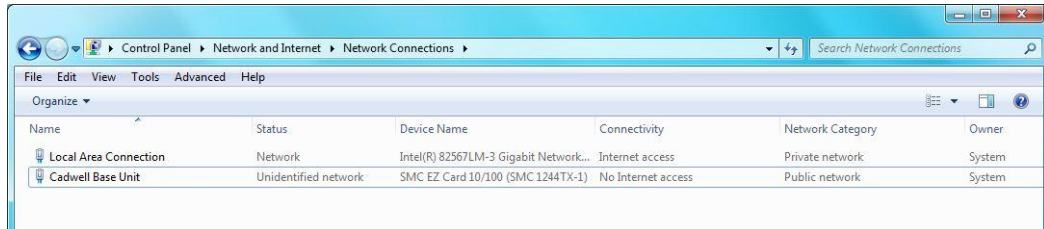


- Uncheck all the protocols in the list except for the **Internet Protocol Version 4 (TCP/IPv4)**.

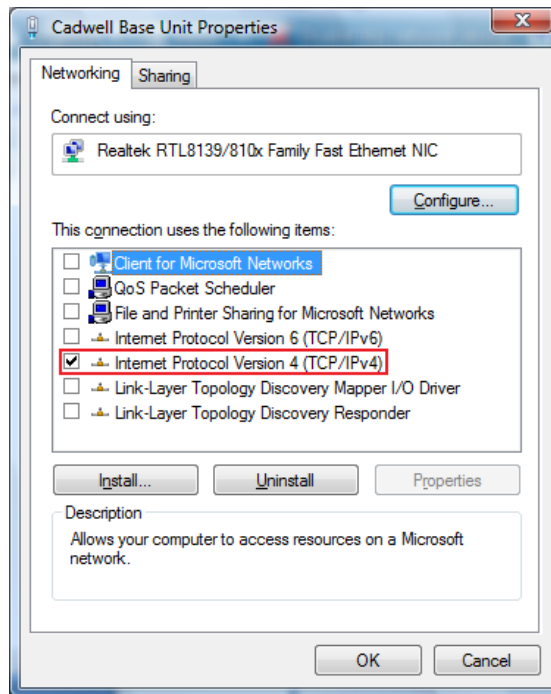

- Then double-click on that protocol and change the radial button from “Obtain an IP address automatically” to **“Use the following address.”**

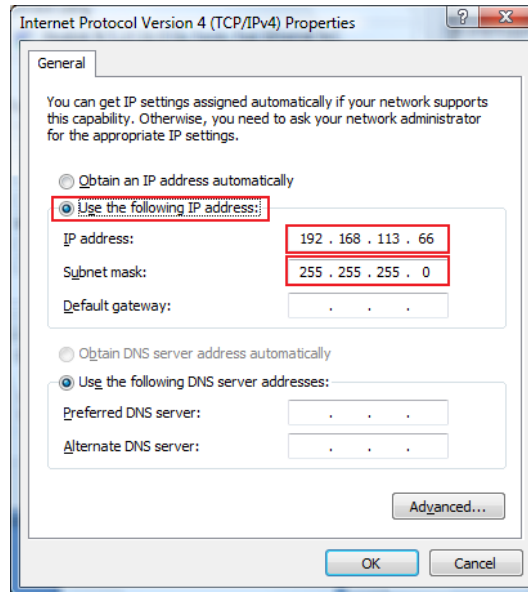

- Input the default Cadwell® IP address. It is **192.168.113.66**. The default Subnet Mask can be used of **255.255.255.0**.
- Once those addresses are added click on the **Advanced** button in the bottom right-hand corner of the screen. Click on the **WINS** tab and uncheck the “**Enable LMHOSTS lookup**” check box and change the NetBIOS setting from Default to “**Disable NetBIOS over TCP/IP**”.

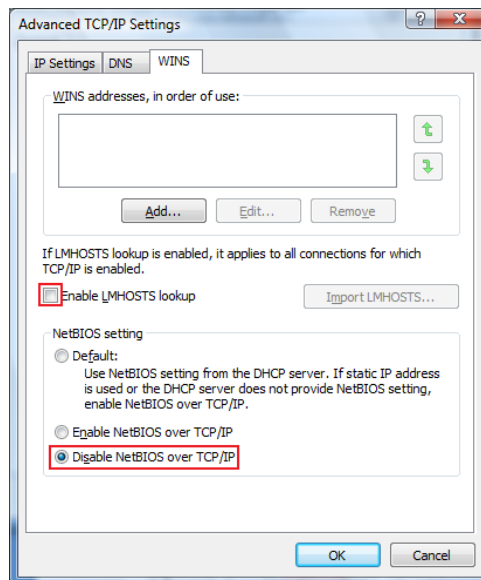


- Click **OK** three times to get back to the network connections page.


- Once again, **right-click** on the LAN or High-Speed Internet connection that will be used to communicate to the base unit and click on **“Properties**.
- Click on the **Configure** button in the upper left corner.



- Click on the **Advanced** Tab then select the **“Speed and Duplex”** listing. *It might also be displayed as “Media Type, Connection Type or Link Speed & Duplex”. Some Windows 7 systems will have a Link Speed tab.*

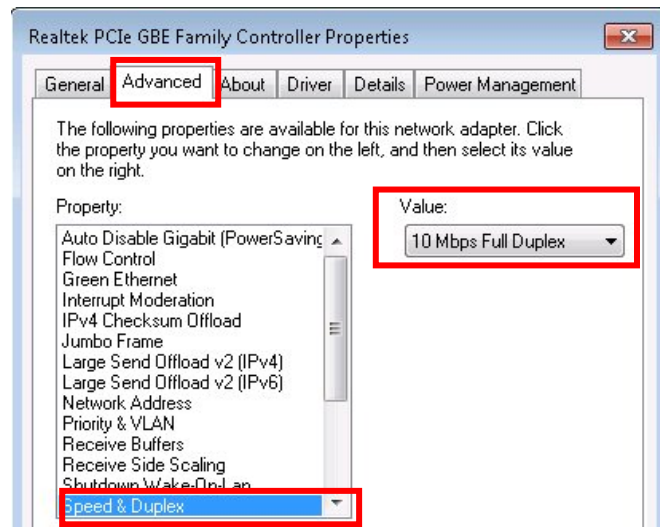


- Click the drop down list on the value right and select **10 Mbps Full Duplex**.
- Click on the **Power Management** tab. **Deselect the “Allow the computer to turn off this device to save power”** check box and then click **OK**. *Note not all network cards will have a power management tab.*

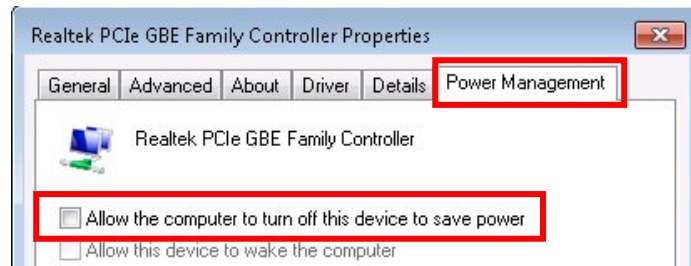


- **Right-click** on the LAN or High-Speed Internet connection that will be used to communicate to the base unit and click on “**Rename.**” Type in **Wave Base** and press **Enter**.

3. Add the Run Command and Printers to the start menu:

- Right-click the **Start Orb** and click “**Properties.**” In the start menu tab click the “**Customize**” button.

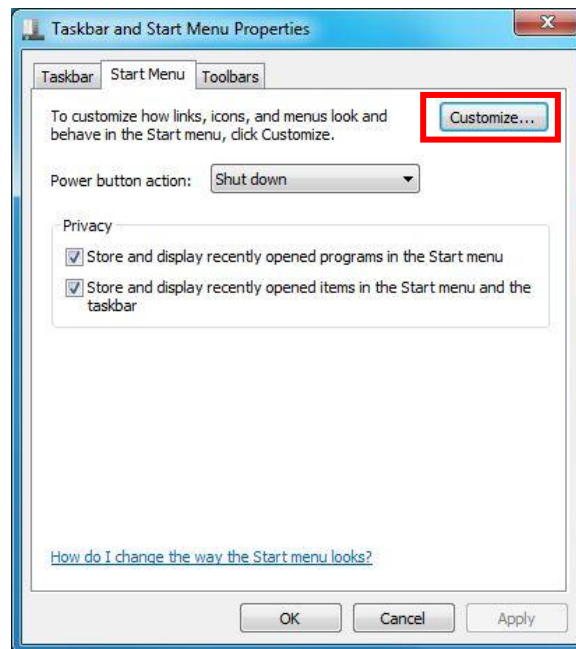

- Check the “**Run Command**” and “**Devices and Printers**” boxes and click **OK** twice.

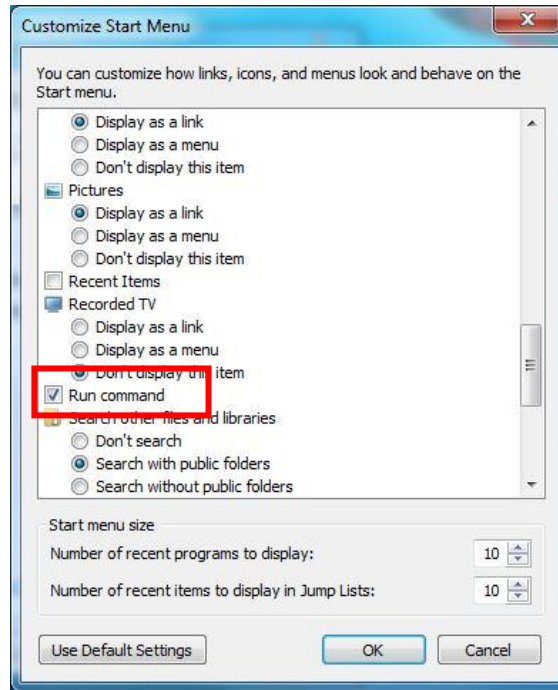

4. Fix the New Program Window so that the QuickReport program will automatically come to the front when launched. This is per the Microsoft Knowledge Base article #886217:


To resolve this issue, modify the value of the ForegroundLockTimeout registry entry in Registry Editor. To do this, follow these steps:

- 1 Click the **Start Orb**, click **Run**, type **regedit** in the **Open** box, and then click **OK**.
- 2 In Registry Editor, locate and then click the following registry subkey:
HKEY_CURRENT_USER\Control Panel\Desktop
- 3 In the right pane, locate and then double-click **ForegroundLockTimeout**.
- 4 Under **Base**, click **Decimal**, type **0** in the **Value data** box, and then click **OK**.
Note The default decimal value for the ForegroundLockTimeout registry entry is set as **200000**.
- 5 Quit Registry Editor.
- 6 **Restart** your computer.


**This must be done for each User Account on the system.*

5. Add the “Computer” and “Network” icons to the desktop:

- Right-click on the desktop and click **Personalize**.
- Click on “**Change Desktop Icons**” under Tasks.

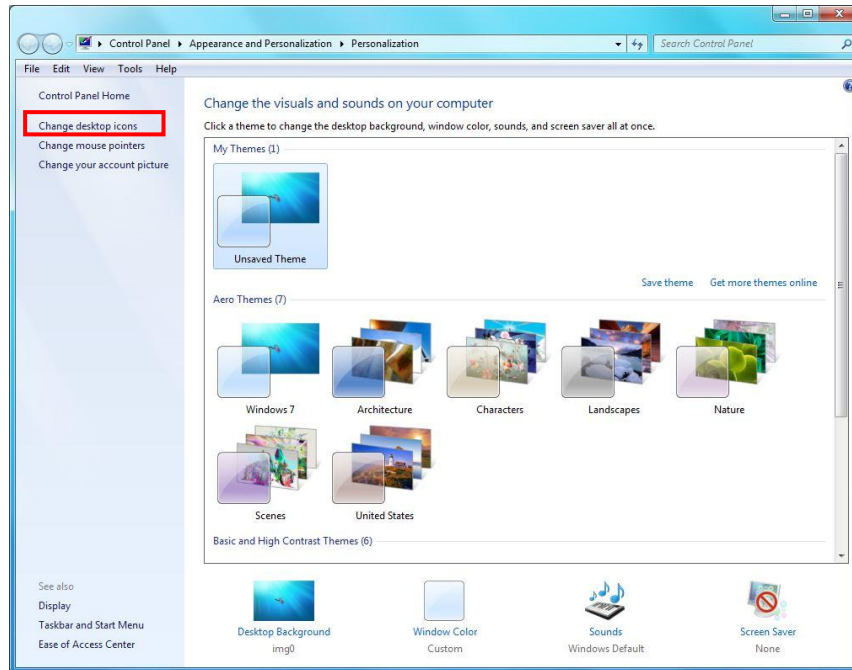


- Check the boxes for the **Computer** and **Network** from the available icons.
- Click **Apply** than Click **OK**.

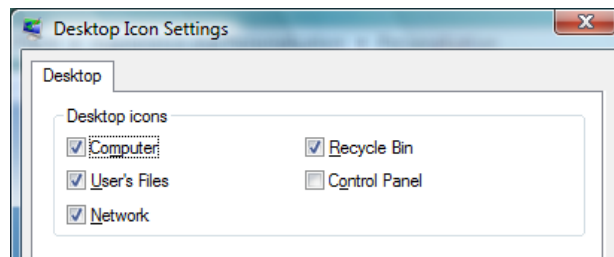

6. Disable password protection on Screen Saver:


- Right-Click anywhere on the desktop and click “**Personalize.**” Click on the **Screen Saver** link in the lower right corner.

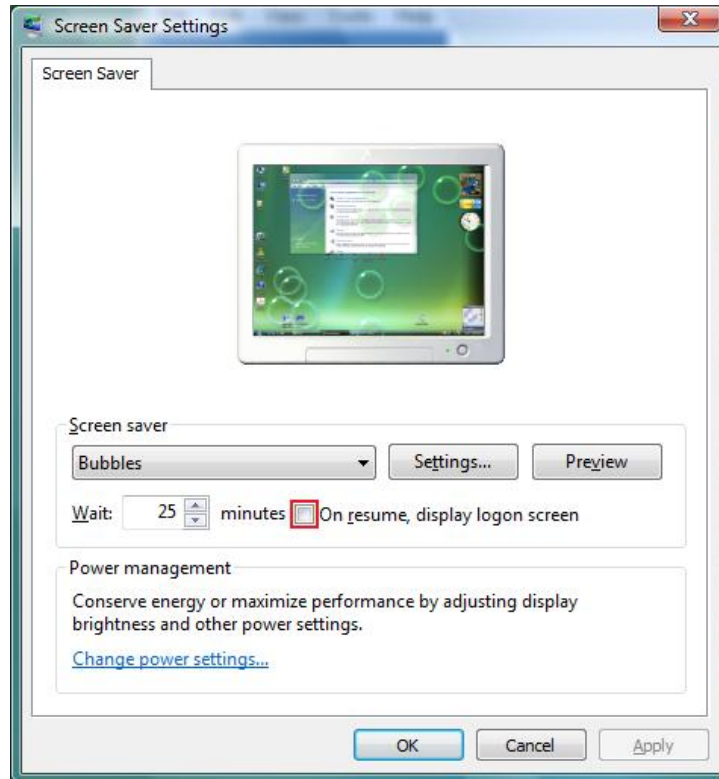


- Then **uncheck** the “**On resume, display the login screen**” and click **OK**.

7. Disable the animation controls and elements within windows:


- Right-click on the **Computer icon** and click **Properties**, then click on the “**Advanced System Settings**” link.

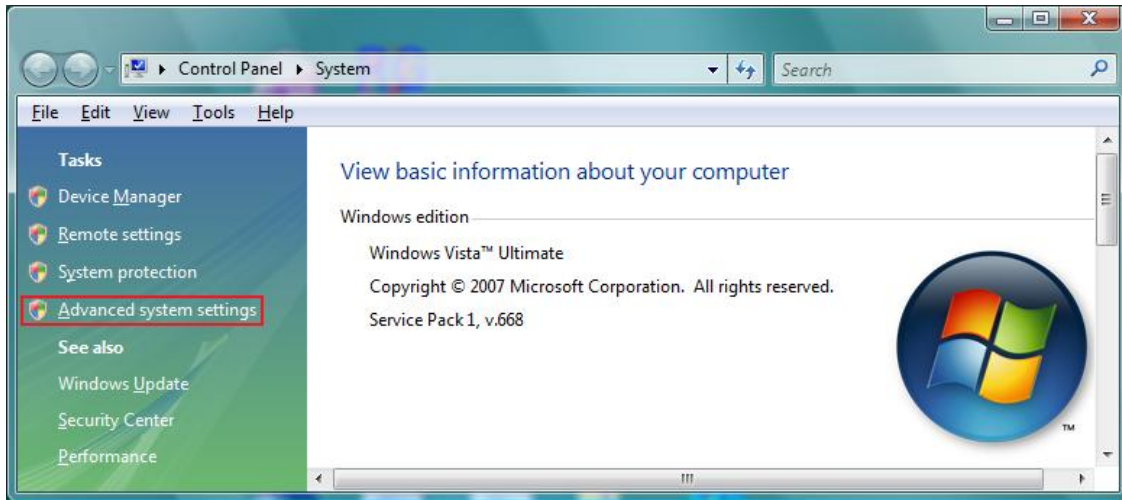

- Click on the **Advanced tab**, then the **Settings** button in the **Performance** section.

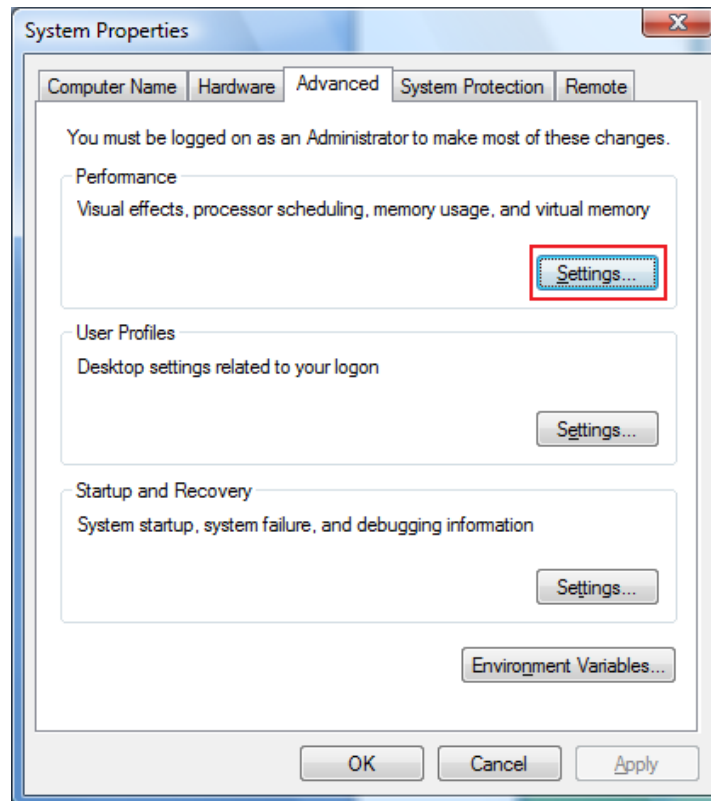


- This will bring up the dialog box that will turn off the animations of buttons. Uncheck the top check box named “**Animate controls and elements inside windows.**”

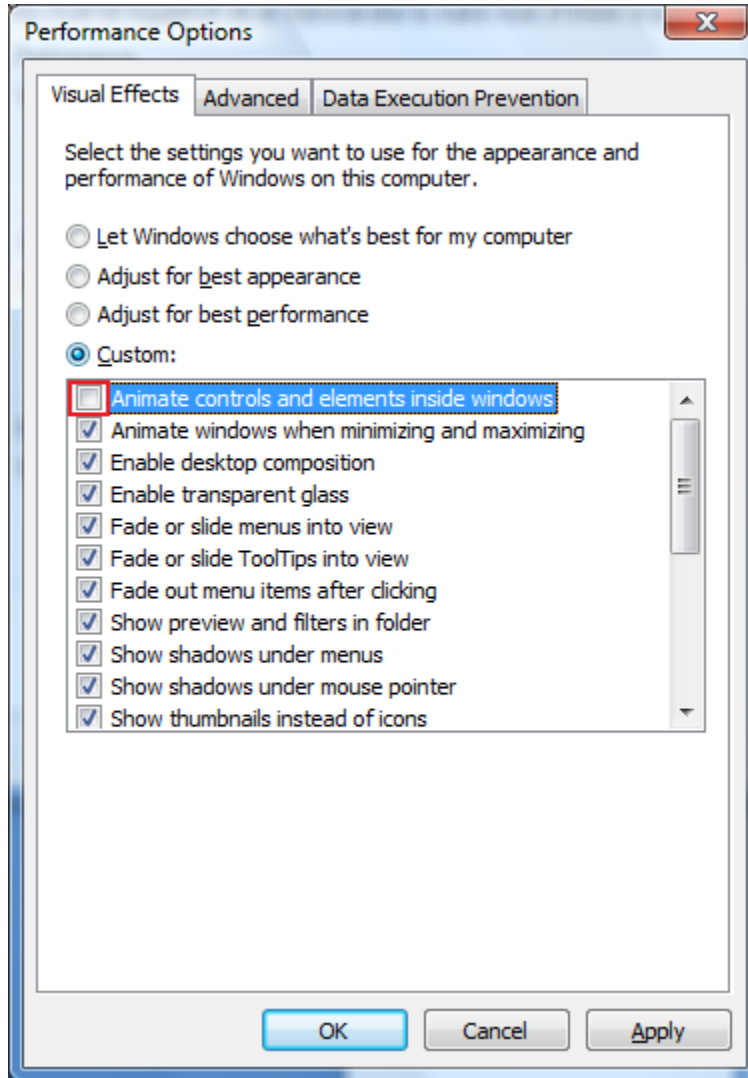

- Once that is done click **Apply** and **OK** out of everything.

8. Enable the Menu Bar in the Explorer window:


- Double-click “**Computer**” then click “**Organize**” then hover over “**Layout**” then click on “**Menu Bar**.”
- The File, Edit, View, Tools and Help dropdowns should now be visible.

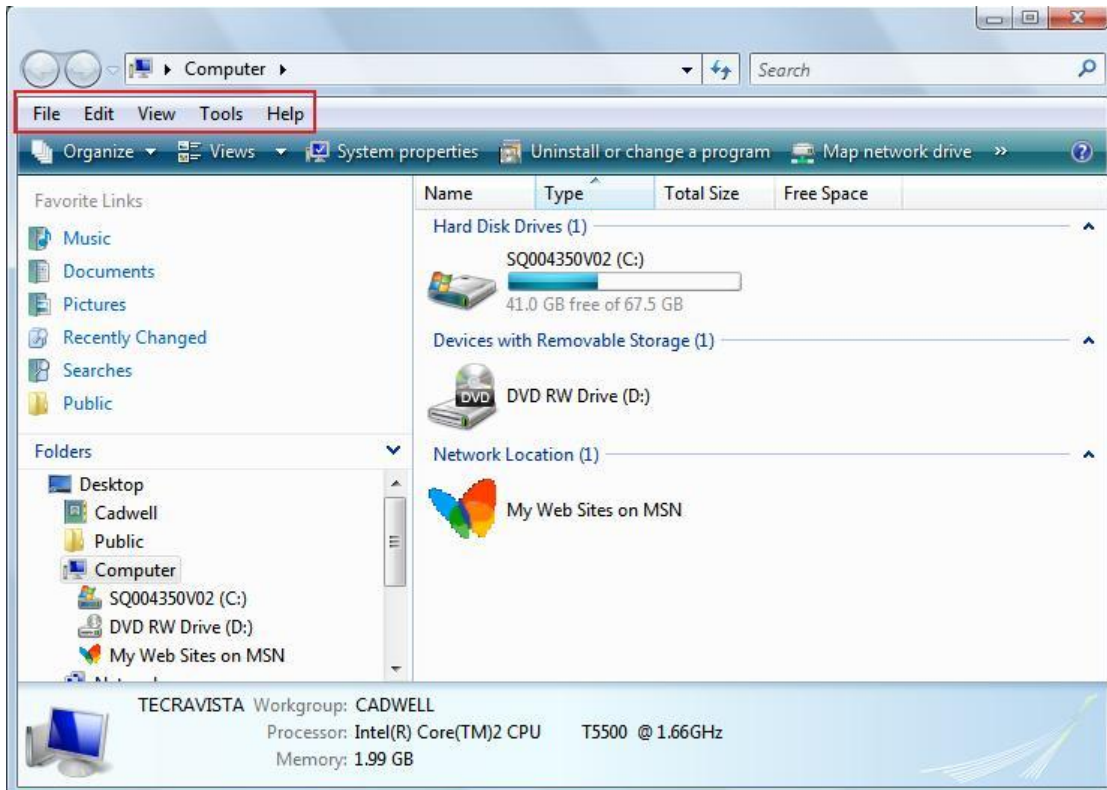


9. Configure the Power Settings for a Laptop or Desktop:




- Click the **Start Orb**.
- Click on the **Control Panel**.
- Change the “**View by:**” to “**Large Icons**” or “**Small Icons**”. *Do not use “Category View”.*
- In the control panel list click on “**Power Options**”.
- Select the **High Performance** option. If you don’t see this option, click on Show Additional Plans.

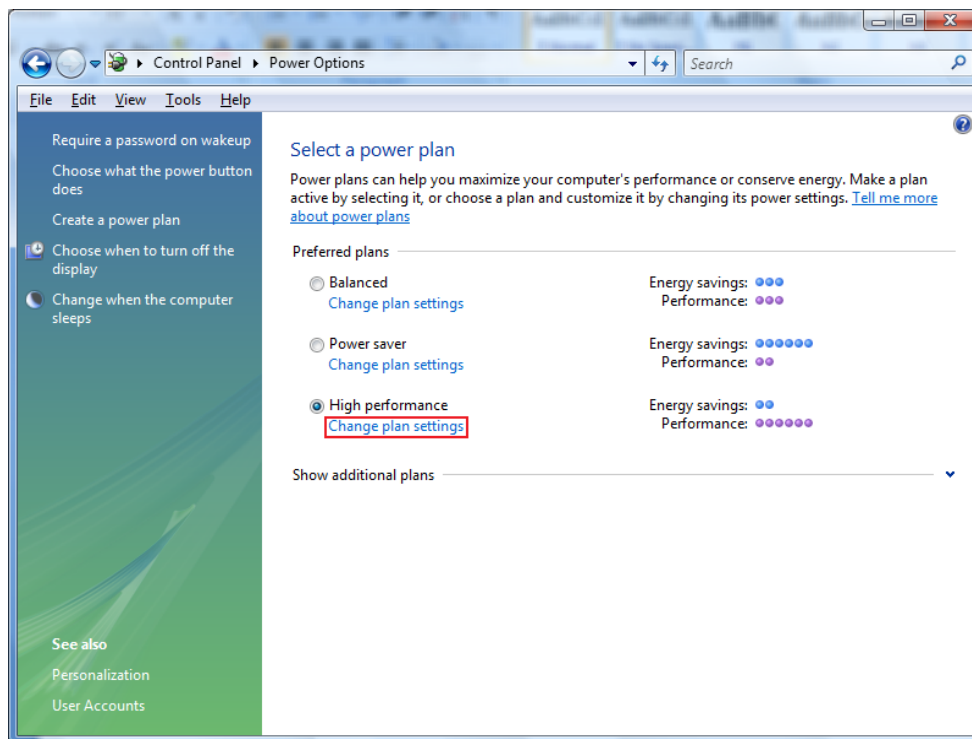

- Click on the “**Change Plan Settings**” link under High Performance.
- Click the dropdown list for both “**Turn off the Display**” and “**Put the Computer to Sleep**” and set them to **NEVER**.

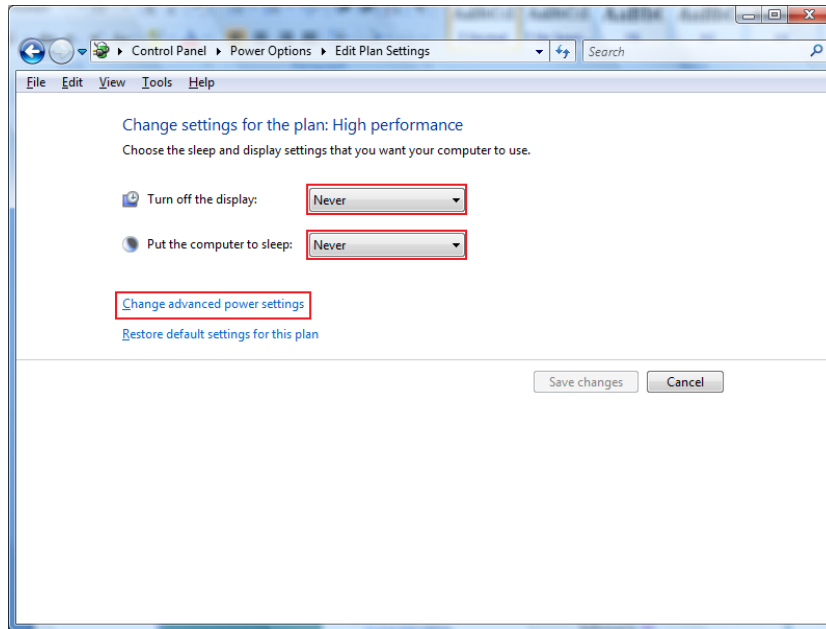

- Next click the “**Change Advanced Power Settings**” link and set the “**Link State Power Management**” setting to **OFF**. That only needs to be done under the **PCI Express** setting. Then click **OK** out of everything to set all the power settings.

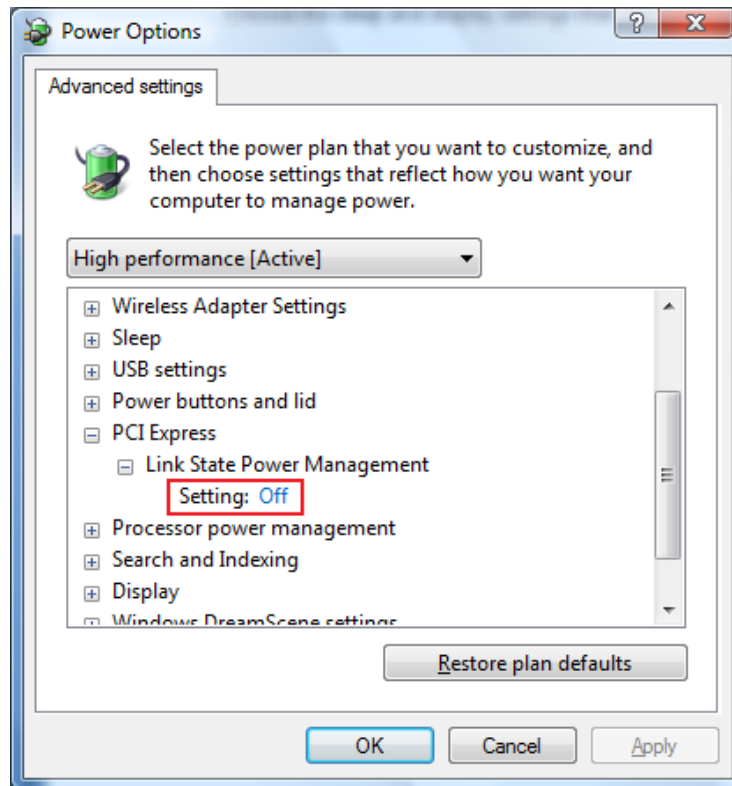

10. Show File Extensions



- Click the **Start Orb**.
- Click on the **Control Panel**.
- Change the “**View by:**” to “**Large Icons**” or “**Small Icons**”. *Do not use “Category View”.*
- Click on “**Folder Options**”
- Click on the **View** tab.
- **Uncheck** the box “**Hide extensions for known file types**”.
- Click **Apply** than **OK**.

11. Configure the Taskbar Properties:

- **Right click** on the **Taskbar** at the bottom of the screen and select **Properties**.
- For the **Taskbar Buttons**, change the setting to “**Combine when taskbar is full**” or “**Never Combine**”.
- Click **Apply**, and then click **OK**.

